

PLEGUI EYELET SCREW PEGS


► **Indication:** The implants of the Plegui range are intended for patella and olecranon fractures treatments by tension band wiring in adults.

Contra-indications:


- Serious vascular deterioration, bone devitalization.
- Pregnancy.
- Acute or chronic local or systemic infections.
- Lack of musculo-cutaneous cover, severe vascular deficiency affecting the concerned area.
- Insufficient bone quality preventing a good fixation of the implants into the bone.
- Muscular deficit, neurological deficiency or behavioral disorders, which could submit the implant to abnormal mechanical strains.
- Allergy to one of the materials used or sensitivity to foreign bodies.
- Serious problems of non-compliance, mental or neurological disorders, failure to follow post-operative care recommendations.
- Unstable physical and/or mental condition.

TECHNICAL FEATURES

► EYELET SCREW PEG

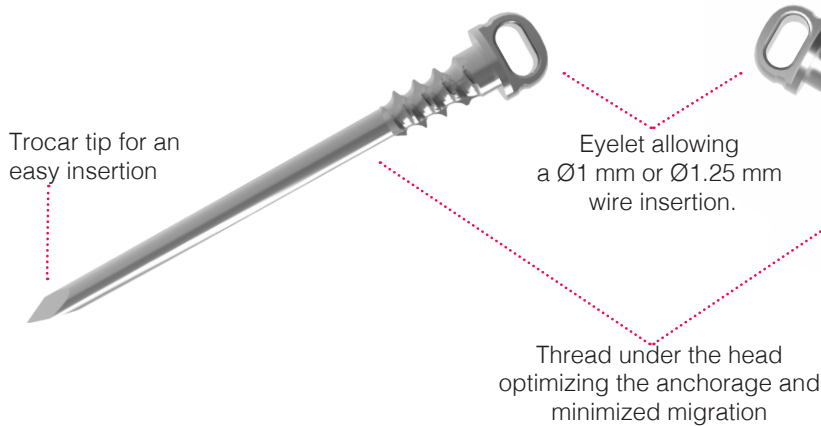


► EYELET SCREW PEG WITH DISTAL THREAD

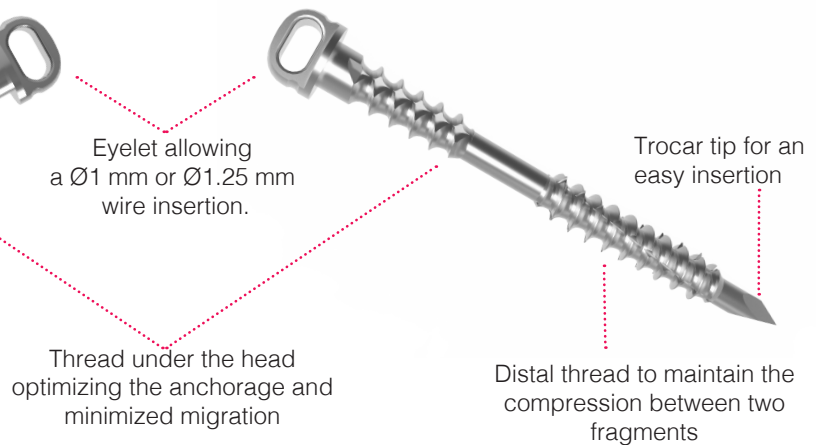

DEDICATED INSTRUMENTATION

ASSEMBLY

PREHENSOR SCREWDRIVER + EYELET SCREW PEG AND EYELET SCREW WITH DISTAL THREAD

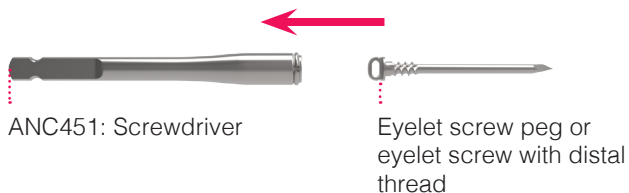


FINAL RESULT




SURGICAL TECHNIQUE


Example: Patella fracture fixation

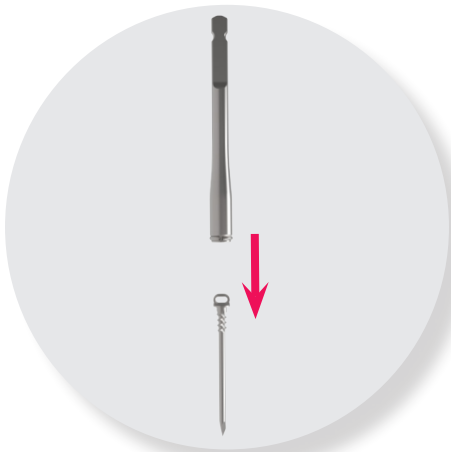

1. Position the eyelet screw peg into the dedicated screwdriver (ANC451).

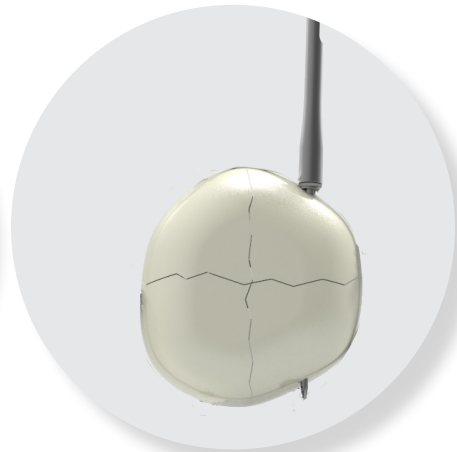

2. Insert the screw through the fracture site using the power tool.

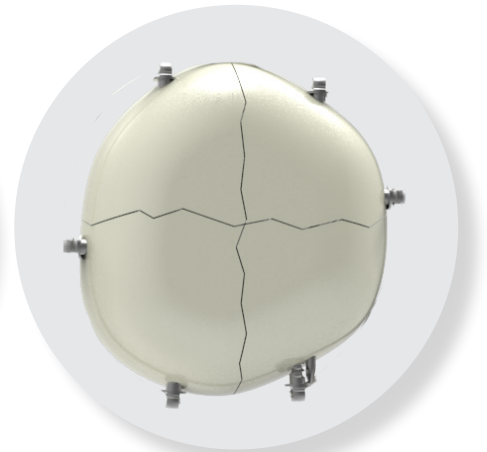

3. According to the fracture type, repeat this procedure all around the patella until the entire fracture reduction is reached.


FINAL RESULT



Once each fragment is repositionned and maintained by the eyelet screws, adjust the orientation of all eyelets (using the assembly : screwdriver + handle).

Run the cerclage wire through all the eyelets all around the patella and then twist the two parts of the wire to perform the compression.

REFERENCES

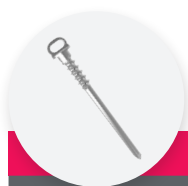


EYELET SCREW PEG - Ø1.8 mm

Ref.	Description
ET1.8L15	Eyelet screw peg - L 15 mm
ET1.8L20	Eyelet screw peg - L 20 mm
ET1.8L25	Eyelet screw peg - L 25 mm
ET1.8L30	Eyelet screw peg - L 30 mm
ET1.8L35	Eyelet screw peg - L 35 mm
ET1.8L40	Eyelet screw peg - L 40 mm
ET1.8L45	Eyelet screw peg - L 45 mm
ET1.8L50	Eyelet screw peg - L 50 mm
ET1.8L55	Eyelet screw peg - L 55 mm
ET1.8L60	Eyelet screw peg - L 60 mm

Remark:

Please note that all implants are also available in sterile packaging. The tube packaging is handy and easy to use. An "ST" code is added at the end of the reference, e.g. "ET1.8L25-ST".

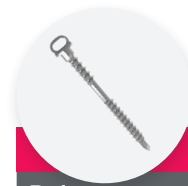

EYELET SCREW WITH DISTAL THREAD - Ø2.8 mm

Ref.	Description
FT2.8L35	Eyelet screw with distal thread - L 35 mm
FT2.8L40	Eyelet screw with distal thread - L 40 mm
FT2.8L45	Eyelet screw with distal thread - L 45 mm
FT2.8L50	Eyelet screw with distal thread - L 50 mm
FT2.8L55	Eyelet screw with distal thread - L 55 mm
FT2.8L60	Eyelet screw with distal thread - L 60 mm

INSTRUMENTATION

Ref.	Description
ANC350	Ø4.5 mm AO quick coupling handle - Size 1
ANC451	Screwdriver

